

OPERATING INSTRUCTIONS

Lift Control with Heat and Massage

MODEL: HC-2802-3

To Operate Lift

Press UP key to raise chair, Press DOWN key to lower chair

To Lock/Lift

Press and hold LOCK/UNLOCK key for approx 3 seconds until red lock icon is lit

To Unlock Lift

Press and hold LOCK/UNLOCK key for approx 5 seconds until green unlock icon is lit

To Operate Heat

Press HEAT key to turn on, Press HEAT key again to turn off

To Operate Massage

Press MASSAGE key to turn on massage, press again to change from WAVE/STEADY/PULSE modes (colored LEDs indicate selection)

To Turn MASSAGE OFF

Press and hold MASSAGE key

Massage Intensity

Press INTENSITY key to choose from LO/MED/HI (colored LEDs indicate selection)

NOTE: The heat and massage is controlled by a 20 minute timer

IMPORTANT SAFETY INSTRUCTIONS

WARNING: To reduce risk of burns, fire, electric shock, or injury to persons:

- READ INSTRUCTIONS BEFORE USING
- BURNS WILL RESULT FROM IMPROPER USE
- NEVER REMOVE COVERS

IMPORTANT: Patients using a pacemaker should consult a physician prior to use.

This appliance should never be left unattended when plugged in. Unplug from outlet when not in use, and before putting on or taking off parts.

Close supervision is necessary when this unit is used by, on, or near children, invalids, or disabled persons.

Use this unit only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer. Keep the cord away from heated surfaces. Do not use outdoors.

Do not operate where aerosol (spray) products are being used or where oxygen is being administered.

Never operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the appliance to a service center for examination and repair.

To disconnect, turn all controls to the off position, then remove plug from outlet.

SAVE THESE INSTRUCTIONS

HC-2802-3

OPERATING INSTRUCTIONS

DANGER - TO REDUCE RISK OF ELECTRIC SHOCK:

Always unplug the unit from the electrical outlet immediately after using.

**Do not reach for an appliance that has fallen into water.
Unplug immediately.**

Never use pins or other metallic fasteners with this appliance.

**Carefully examine the covering before each use. Discard the
appliance if the covering shows any sign of deterioration, such
as checking, blistering, or cracking.**

Keep dry - Do not operate in a wet or moist environment.

