

BACKCYCLER COMPONENTS



NOTES:


Modes – The BackCycler is equipped with two modes of operation – Static Lumbar Support and Auto Cycle Mode.

Static Lumbar Support - If not in Auto Cycle Mode, the unit will maintain the air pressure set by the user.

Auto Cycle Mode - One cycle of the BackCycler includes the time to inflate the unit to the desired set point and completely deflate the unit.

Timer - A 90 minute timer is included in the Auto Cycle Mode of the BackCycler. The unit will power off after 90 minutes of cycling without any user interaction. If the unit times out, press the Cycle key to resume cycling at the previous set points.

OPERATING INSTRUCTIONS

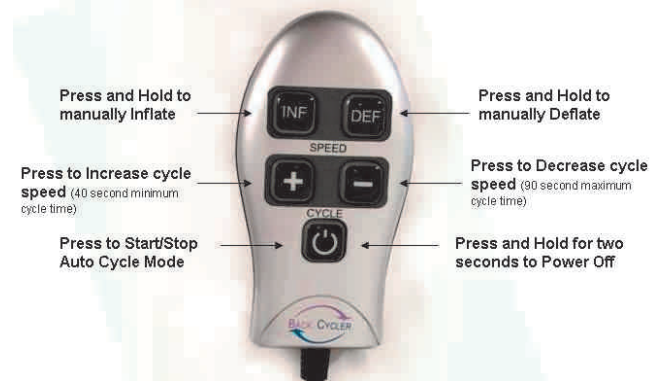

Note: When the unit is initially plugged in, all lights will turn on while the unit calibrates (15 seconds)

1. Press any key to power on.
2. Press INF or DEF key to manually adjust the system pressure to the desired set point.
 Note: The BackCycler will maintain the static pressure set in Step 2 above until the pressure is manually changed or the Auto Cycle Mode is started.
3. Press the Cycle key to commence Auto Cycle Mode.
 Notes: a. The system will cycle to the set point selected in Step 2 above. To adjust the cycle pressure, repeat Steps 2 and 3.
 b. To stop Auto Cycle and resume Static Lumbar Support, press the Cycle key.
4. Press the Cycle Speed keys (+ and -) to adjust the Auto Cycle time.
 Note: The user may adjust the cycle time from 40 seconds to 90 seconds using the (+) and (-) keys while in Auto Cycle Mode.
5. Press and hold the Cycle key for 2 seconds to power off.

Function	Maximum (or Set Point)	Minimum
Static Pressure	5 psi	0 psi
Cycle Time	90 seconds	40 seconds
Auto Cycle Inflate Pressure	5 psi	1.5 psi
Timer	90 minutes (Auto Cycle Mode only)	

SAVE THESE INSTRUCTIONS

IMPORTANT SAFETY INSTRUCTIONS

WARNING: To reduce risk of burns, fire, electric shock, or injury to persons:

- READ INSTRUCTIONS BEFORE USING
- BURNS MAY RESULT FROM IMPROPER USE
- NEVER REMOVE CONTROL UNIT COVERS

IMPORTANT: Patients using a pacemaker should consult a physician prior to use.

Unplug from outlet before adding on or taking off parts.

Close supervision is necessary when this unit is used by, on, or near children, invalids, or disabled persons.

Use this unit only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer. Keep the cord away from heated surfaces. Do not use outdoors.

Do not operate where aerosol (spray) products are being used or where oxygen is being administered.

Never operate this unit if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the unit to a service center for examination and repair.

To disconnect, turn all controls to the off position, then remove plug from outlet.



BackCycler™

OWNERS MANUAL AND OPERATING INSTRUCTIONS

Model BC-1000A-OEM BackCycler™

Get Back-in-Motion with the BackCycler™

BackCycler™ features:

- Continuous Passive Motion Lumbar Support System
- Dynamic Motion Therapy to Relieve Stiffness & Fatigue
- Effortless Back Care
- Increases Lower Back Flexibility

US Patent Numbers: 4,981,131; 5,624,383
5,637,076; 6,544,203